English

For Use Under an Emergency Use Authorization Only (US)

For *in vitro* diagnostic use only.

Intended Use

The Xpert® Xpress Nasopharyngeal Sample Collection Kit is designed to collect, preserve, and transport nasopharyngeal swab specimens and to preserve and transport nasal aspirate/wash specimens containing viruses from patients with signs and symptoms of respiratory infection prior to analysis with Xpert SARS-CoV-2 tests.

Summary and Explanation

One of the routine procedures in the diagnosis of infections caused by respiratory viruses involves the collection and safe transportation of biological samples. This can be accomplished using the Xpert Xpress Nasopharyngeal Sample Collection Kit. This system includes the Xpert Viral Transport Medium that is stable at room temperature and can sustain viability (and infectivity) of respiratory viruses during transit to the testing laboratory or testing area. The formulation of the Xpert Xpress Nasopharyngeal Sample Collection Kit medium includes protein for stabilization, antibiotics to minimize bacterial and fungal contamination, and a buffer to maintain a neutral pH. The Xpert Xpress Nasopharyngeal Sample Collection Kit consists of one capture-cap tube containing medium and one peel pouch incorporating a sterile nylon flocked nasopharyngeal specimen collection swab with a scored shaft for easy breakage. The capture-cap is designed to secure the shaft of the swab to the cap, eliminating the use of forceps to remove the swab in the laboratory or testing area.

Principles of the Procedure

The Xpert Viral Transport Medium consists of modified Hank's balanced salt solution supplemented with bovine serum albumin, cysteine, gelatin, sucrose, and glutamic acid. The pH is buffered with HEPES buffer. Phenol red is used to indicate pH. Vancomycin, amphotericin B, and colistin are incorporated in the medium to inhibit growth of competing bacteria and yeast. The medium is isotonic and non-toxic to mammalian host cells. The presence of sucrose acts as a cryoprotectant, which aids in the preservation of respiratory viruses if specimens are frozen (-70 °C) for prolonged storage.

Reagents

Materials Provided

Xpert Xpress Nasopharyngeal Sample Collection Kit (quantity: 100)

Each kit includes a sealed package containing:

- 1 individually wrapped sterile nylon flocked nasopharyngeal swab
- 1 Xpert Viral Transport Medium tube containing:
 - Hanks' Balanced Salts
 - Bovine Serum Albumin
 - L-Cysteine
 - Gelatin
 - Sucrose
 - L-Glutamic Acid
 - HEPES Buffer
 - Vancomycin
 - Amphotericin B
 - Colistin
 - Phenol Red
 - pH 7.3 ± 0.2 @ 25°C

Materials Needed but Not Provided

- Disposable gloves
- Adhesive labels or permanent markers for recording specimen information

Warnings and Precautions

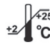
- Do not re-sterilize unused swabs.
- Do not re-pack.
- Not suitable to collect and transport microorganisms other than respiratory viruses.
- Not suitable for any other application than the intended use.
- Do not use if the swab is visibly damaged (i.e., if the swab tip is broken).
- Do not ingest the medium.
- Do not use the Xpert® Viral Transport Medium for premoistening or prewetting the applicator swab prior to collecting the sample or for rinsing or irrigating the sampling sites.
- Do not bend the swab prior to specimen collection.
- Specimen collection and handling procedures require specific training and guidance.
- Observe approved biohazard precautions and aseptic techniques.
- To be used only by adequately trained and qualified personnel.
- All biological specimens should be considered potentially infectious and handled in a manner that prevents infection of laboratory personnel.
- Sterilize all biohazardous waste, including specimens, containers, and media after their use.
- Directions should be read and followed carefully.

Chemical Hazards

According to Regulations (EC) No. 1272/2008 CLP and (OSHA) CFR 29 1910 1220, this material is not considered hazardous.

Kit Storage Requirements

 The product should be stored in its original container at 2-25°C. Do not overheat. Do not incubate or freeze prior to use. Improper storage will result in a loss of efficacy. Do not use after the expiration date, which is clearly printed on the outer box label.

Product Deterioration

The Xpert Xpress Nasopharyngeal Sample Collection Kit should not be used if (1) there is evidence of damage or contamination to the product, (2) there is evidence of leakage, (3) the color of the medium has changed from light orange-red, (4) the expiration date has passed on any of the components, (5) the swab pouch is open, or (6) there are other signs of deterioration on any of the components.

Limitations

- Specimens should be handled aseptically.
- The Xpert Xpress Nasopharyngeal Sample Collection Kit is intended for use as a collection and transport medium for respiratory viruses only.
- The Xpert Xpress Nasopharyngeal Sample Collection Kit is intended to be used with the medium tube and swab provided in the kit. The use of tubes of medium or swabs from any other source could affect the performance of the product.

Specimen Collection and Handling

- Nasopharyngeal specimens can be collected following the user institution's standard procedures and placed into the Xpert Viral Transport Medium (3 mL tube with transport media).
- Samples can be stored at 15-30 °C for up to 8 hours and at 2-8 °C for up to 7 days before processing with the Xpert Xpress SARS-CoV-2 test.
- Proper specimen collection, storage, and transport are critical to the performance of this test.

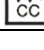
Specimen Collection Procedure

Proper specimen collection from the patient is extremely critical for successful isolation and identification of infectious organisms. Highest viral titers are present during the acute illness.

Note: The nylon flocked swab should not be bent prior to specimen collection.

1. Collect the specimen with the swab. Take care not to touch the tip of the swab and not to lay it down.
2. Aseptically remove the cap from the tube.
3. Insert the swab into the tube with the transport medium.
4. Break the swab shaft at the pre-scored line by bending it against the tube wall.
5. Replace the cap onto the tube and close tightly.
6. Label with appropriate patient information.
7. Send to the laboratory or testing area for immediate analysis.

Table of Symbols

Symbol	Meaning
	Catalog number
	CE marking – European Conformity
	<i>In vitro</i> diagnostic medical device
	Contains sufficient for <n> tests
	Sterilized using ethylene oxide
	Authorized representative in the European Community
	Do not reuse
	Manufacturer
	Temperature limitation
	Batch code
	Caution
	Consult instructions for use
	Expiration date
	Biological risks
	Handle with care
	For prescription use only
	Country of Manufacture

Technical Support Contact

Region	Telephone	Email
US	+1 888 838 3222	techsupport@cepheid.com
France	+33 563 825 319	support@cepheideurope.com

Contact information for all Cepheid Technical Support offices is available on our website: www.cepheid.com/en/CustomerSupport.

Cepheid
904 Caribbean Drive
Sunnyvale, CA 94089
USA
Phone: +1 408 541 4191
Fax: +1 408 541 4192

Cepheid Europe SAS
Vira Solelh
81470 Maurens-Scopont
France
Phone: +33 563 825 300
Fax: +33 563 825 301

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