


# Rectal/Perirectal Specimen Collection


1

Use Cepheid Collection Device #900-0370 to collect the specimen.

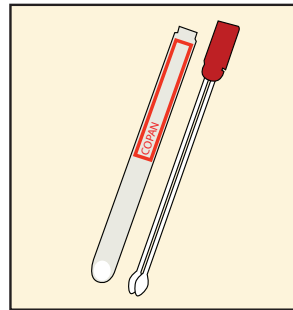


3

Place the swab pair back into the transport tube.

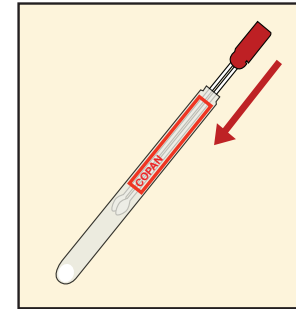


4

Swabs in the transport tube can be stored at 15-28 °C for up to five days.

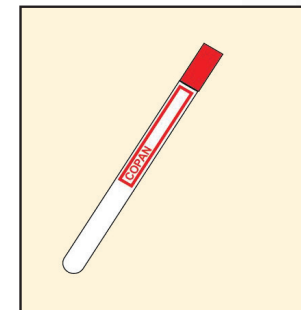

## Rectal Swab Sample

Carefully insert both swab tips approximately 1 cm beyond the anal sphincter and rotate gently.

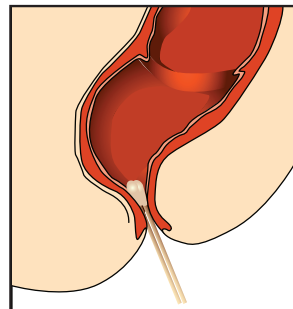

2

OR

## Perirectal Swab Sample

Carefully insert both swab tips no more than 1 cm into the anal opening before the anal sphincter and rotate gently.

